
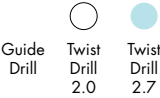








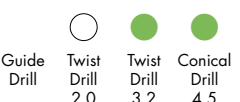
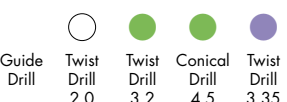








Drilling sequence overview

OsseoSpeed™ Implants	Drilling protocol – standard	Drilling protocol – dense bone
 <p>OsseoSpeed™ 3.0 S</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 2.7</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 2.7 Cortical Drill 3.0 Twist Drill 2.85</p>
 <p>OsseoSpeed™ 3.5 S</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 3.2</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 3.2 Cortical Drill 3.5 Twist Drill 3.35</p>
 <p>OsseoSpeed™ 4.0 S</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 3.2 Twist Drill 3.7</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 3.2 Twist Drill 3.7 Cortical Drill 4.0 Twist Drill 3.85</p>
 <p>OsseoSpeed™ 4.5</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 3.2 Conical Drill 4.5</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 3.2 Conical Drill 4.5 Twist Drill 3.35</p>
 <p>OsseoSpeed™ 5.0</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 3.2 Twist Drill 3.7 Conical Drill 5.0</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 3.2 Twist Drill 3.7 Conical Drill 5.0 Twist Drill 3.85</p>
 <p>OsseoSpeed™ 5.0 S</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 3.2 Twist Drill 3.7 Twist Drill 4.2 Twist Drill 4.7</p>	 <p>Guide Drill Twist Drill 2.0 Twist Drill 3.2 Twist Drill 3.7 Twist Drill 4.2 Twist Drill 4.7 Cortical Drill 5.0 Twist Drill 4.85</p>

Single Patient Drill Kits

Stainless steel
Sterile.
Laser etched depth indication.

Includes:
Guide Drill
SP Twist Drill Ø 2.0
SP Twist Drill Ø 3.2



Short, 8–13 mm
REF 22898



Long, 8–19 mm
REF 22899

Single Patient Drill Kit 6–13 mm

Stainless steel
Sterile.
Laser etched depth indication.

Includes:
SP Twist Drill Ø 2.0, 6–13 mm
SP Twist Drill Ø 3.2, 6–13 mm



REF 24737*

Note: SP Drills included in kits are not available as separate items.

Single Patient Drills

Stainless steel
Sterile.
Laser etched depth indication.

	REF				
SP Twist Drill Ø 3.7, 6–13 mm	24199*				
SP Twist Drill Ø 3.85, 6–13 mm	24734*				
	REF	Short, 8–13 mm	REF	Long, 8–19 mm	
SP Twist Drill Ø 3.35	22908		22909		
SP Twist Drill Ø 3.7	22910		22911		
SP Twist Drill Ø 3.85	22912		22913		
SP Twist Drill Ø 4.2	22957		22958		
SP Twist Drill Ø 4.7	22959		22960		
SP Twist Drill Ø 4.85	22961		22962		
	REF	Short shaft	REF	Long shaft	
SP Pilot Drill Ø 2.0/3.2, 6–13 mm	24735*				
SP Pilot Drill Ø 2.0/3.2	22904		22905		
SP Pilot Drill Ø 3.2/3.7	22906		22907		
SP Pilot Drill Ø 3.7/4.2	22963				
SP Cortical Drill Ø 3.5	22914		22915		
SP Cortical Drill Ø 4.0	22916		22917		
SP Cortical Drill Ø 5.0	22964				
SP Conical Drill Ø 4.5	22921		22922		
SP Conical Drill Ø 5.0	22923		22924		

* For OsseoSpeed™ 4.0 S – 6 mm