

Implant installation guide

Preparation



1. Peel off the perforated section of the label and use it for documentation and/or communication with your restorative partner.



2. Open and slide the sterile inner container onto a sterile surgical area. Lift the cap to expose the implant.




3. Attach the appropriate Implant Driver to the Contra Angle. Make sure that the driver is properly seated. Pick up the implant from the inner container.

Installation



4. Install the implant with a contra angle at low speed (25 rpm) under profuse irrigation. Set the maximum torque to 35 Ncm. Let the implant work its own way into the osteotomy and avoid applying unnecessary pressure.

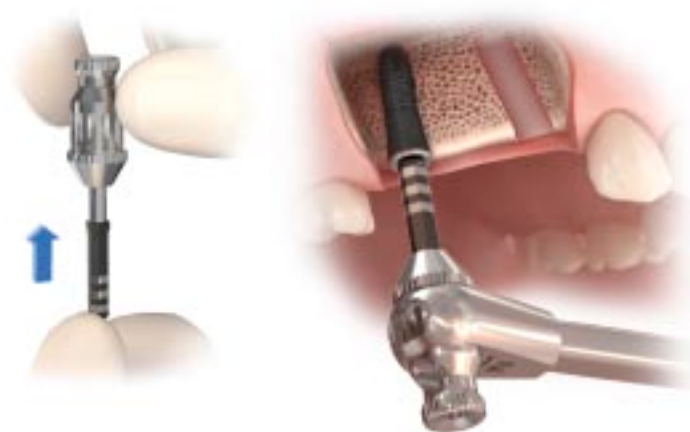

5. The Ratchet Wrench, in combination with the Driver Handle, may be used for the final seating of the implant. Use light finger force when leveling the implant. A too high torque indicates that the implant needs to be retrieved to facilitate additional drilling.



6. Position the implant at the marginal bone level or slightly below. Position one of the flat surfaces of the Implant Driver buccally to facilitate optimal placement of the chosen abutment. Release the Implant Driver from the implant by shifting it slightly from side to side.